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# CONTENTS

Safety precautions	31
Introduction	32
Precautions before use	33
Functions	33
Usage	34
Practical considerations when taking a temperature	34
How to take a temperature	35
Important information when taking a temperature	35
Description	36
Setting and function	37
Instructions for calibration	38
Changing the batteries	39
Technical characteristics	40
Troubleshooting	41
Warranty	43

The manufacturer reserves the right to alter the specification of the product without prior notification.

## SAFETY PRECAUTIONS

- Follow the maintenance advice stipulated in this instruction manual.
- This device may be used for professional purposes or for personal home use.
- This device must only be used in an ambient temperature range of between 10°C and 40°C.
- This device must always be kept in a clean, dry area.
- Do not expose this thermometer to electric shocks.
- Do not expose this thermometer to extreme temperature conditions of  $> 50^{\circ}\text{C}$  or  $< -20^{\circ}\text{C}$ .
- Do not use this device in relative humidity higher than 85%.
- The protective glass over the lens is the most fragile part of the thermometer.
- Do not touch the glass of the infrared lens with your fingers.
- Clean the glass with a cotton bud lightly moistened with 95% alcohol.
- Do not expose the thermometer to sunlight or to water.
- Never drop the device.
- Do not attempt to repair this device yourself. If you have any problem, please, contact authorised service department.

# INTRODUCTION

The non-contact infrared thermometer Helpmation has been developed by using the latest infrared technology. This technology allows temporal artery (TA) temperature to be taken at a distance of about 3-5 cm away from the forehead. It has been demonstrated that this method of TA temperature measurement is more precise than the tympanic thermometry and better tolerated than rectal thermometry (1). However, as with other types of thermometer, it is essential to use the thermometer properly in order to obtain reliable and stable results.

(1) „Greenes D, Fleischer G. Accuracy of a Noninvasive Temporal Artery Thermometer for Use in Infants. Arch Pediatr Adolesc Med 2001, 155:376.“

## NORMAL TEMPERATURES ACCORDING TO AGE

Age	°C	°F
0-2 years	36.4-38.0	97.5-100.4
3-10 years	36.1-37.8	97.0-100.0
11-65 years	35.9-37.6	96.6-99.7
> 65 years	35.8-37.5	96.4-99.5

## PRECAUTIONS BEFORE USE

The thermometer Helpmation is pre-set at the factory. It is not necessary to calibrate the device when starting.

In order to obtain reliable and stable results, you are advised each time there is a significant change in the ambient temperature due to a change in environment, to allow the thermometer to acclimatize to this ambient temperature for 15 to 20 minutes before using it. It is important to allow 1 minute interval between two measurements.

## FUNCTIONS

1. Special design to take the Human Body Temperature from a distance.
2. Reliable and stable measurement, thanks to the advantage Infrared Detection System.
3. Audible alarm if temperature is too high.
4. Memorize the last 32 temperature measurements.
5. Backlit LCD display.
6. Temperature can be displayed in either °C or °F.
7. Automatic power-off (30 seconds) to save energy.
8. Longevity use (40.000 reading).

The thermometer can also be used to measure the temperature of a baby-bottle or bath, or room temperature (by using the Surface Temp or Room function). This function is in accordance with the Directive 89/339/EEC Electromagnetic compatibility.

## **USAGE**

1. Install batteries (please see Changing the batteries)
2. For the first use or when inserting new batteries wait 10 min. for the warm-up of the device.
3. Aim towards the forehead, from a distance of 3-5 cm, press the measuring button SCAN, the temperature is displayed in 1 second.
4. Before taking the temperature, make sure to remove hair and perspiration from the forehead.

## **PRACTICAL CONSIDERATIONS WHEN TAKING A TEMPERATURE**


In order to ensure that precise and accurate temperature measurements are obtained, it is essential that each user has received adequate information. Temperature should be taken in a neutral context. The patient must not have undertaken vigorous physical activity prior to taking the temperature.

Be aware of physiological variations in temperature which must be taken into consideration when evaluating the results:

- temperature increases by  $0,5^{\circ}\text{C}$  from 6 am to 3 pm
- women have a temperature that is higher, on average, by around  $0,2^{\circ}\text{C}$
- women's temperature also varies in accordance with their ovarian cycle. It rises by  $0,5^{\circ}\text{C}$  in the second half of the cycle and at the early stages of pregnancy
- when sitting, temperature is lower by about  $0,3 - 0,4^{\circ}\text{C}$  than when standing

## HOW TO TAKE A TEMPERATURE

Aim at the forehead, over the right temporal region, from a distance of about 3-5 cm, press the button SCAN and the temperature is instantly displayed.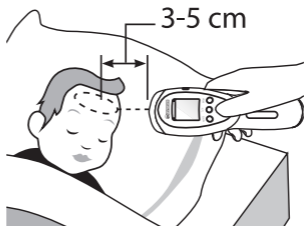

The reliability of the measurement cannot be guaranteed if the temperature is measured over another part of the body (e.g. arm, torso ...).

## IMPORTANT INFORMATION WHEN TAKING A TEMPERATURE


Please observe the following before any temperature measurement to ensure a stable and reliable result:

- push back hair from the forehead
- wipe away any perspiration and make-up from the forehead
- avoid any drafts (e.g. from nasal specs, air conditioning ...)
- allow a 1 minute interval between two measurements
- each time there is a significant change in the ambient temperature due to a change in environment, to allow the thermometer to acclimatise to this ambient temperature for at least 15 minutes before using it

# DESCRIPTION

## LCD DISPLAY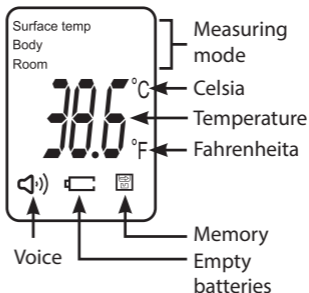

# SETTING AND FUNCTION

## Button SCAN

When the thermometer is off, press the SCAN button once and the thermometer turns on (LCD display lights up). Pressing the SCAN button to measure the desired temperature.

## Button MODE

- Press the MODE button and the display mode is displayed **Body** - to measure the body temperature
- Press the MODE button again and the display shows **Surface Temp** - for measuring the temperature of baby bottles or baths
- Press the MODE button for the third time and the display shows **Room** - for room temperature

Note: The default setting on the thermometer mode BODY

## Button °C/°F

Press °C/°F change measurement from degrees Celsius to Fahrenheity and vice versa

## Button MEM

Press the MEM button, the display shows the last measured value from newest to oldest (memory thermometer in the last 32 measurements).

Pressing MEM displays each temperature thermometer from memory.

## Button LANGUAGE

Press LANGUAGE to enable or disable the audio guide thermometer.

### NOTICE!

**To obtain the internal temperature of the body, always use the „BODY“ mode. Please, make sure to select „Surface temp“ mode for an external area reading.**

## CALIBRATION OF THE THERMOMETER

Thermometer factory-set. Depending on various skin types and thickness, there may be temperature differences.

Press the MODE and °C/°F buttons at the same time for 3 seconds. The screen will then display „F-4“.

Press MODE button to increase 0,1 °C, press °C/°F button to reduce 0,1 °C.

Press MEM button to save the setting.


In the cases of seasonal or environmental changes a verification and adjustment should be carried out.

## CHANGING THE BATTERIES

When the LCD screen displays „Battery“ icon, the batteries are used. Open the lid and change the batteries, taking great care with the correct positioning. („+“ on battery to „+“ in battery compartment and „-“ on battery to „-“ in battery compartment).

A mistake with this could cause damage of the device and compromise the guarantee of your thermometer.

Never use rechargeable batteries. Use only batteries for single usage.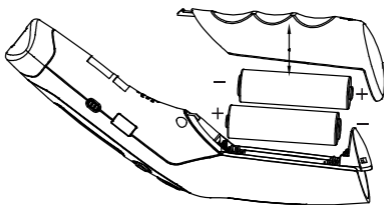

# TECHNICAL CHARACTERISTICS

Normal using condition:

Ambient temperature: 10°C - 40°C (50°F - 104 °F)

Relative humidity: < 85%

Model	RC008
Dimensions	175 x 50 x 52 mm
Weight	148 g (including batteries)
Power (batteries)	DC3 V (2 pcs typ AA, LR6)
Consumption	< 300 mW
Measuring distance	3-5 cm
Automatic power-off	30 sec.

Measuring range:

In Body mode 32°C - 42,9 °C (89,6 °F - 109,2 °F)

In Surface Temp mode 0 °C - 60 °C (32 °F - 140 °F)

In Room mode 0 °C - 60 °C (32 °F - 140 °F)

Precision:

36°C ~ 39°C (96,8 °F ~ 102,2 °F) =  $\pm 0,2^{\circ}\text{C}$  ( $\pm 0,4^{\circ}\text{F}$ )

According to ASTM Standard E1965-1998 (2009)

## Precision

From 34°C to 35,9°C = $\pm 0,3^{\circ}\text{C}$ From 93,2°F to 96,6 °F = $\pm 0,6^{\circ}\text{F}$	According to ASTM Standard E1965-1998 (2009)
From 36°C to 39°C = $\pm 0,2^{\circ}\text{C}$ From 96,8°F to 102,2 °F = $\pm 0,4^{\circ}\text{F}$	
From 39°C to 42,5°C = $\pm 0,3^{\circ}\text{C}$ From 102,2°F to 108,5 °F = $\pm 0,6^{\circ}\text{F}$	


The thermometer takes temperature readings below 32 °C or above 42,9 °C (89,6 °F to 109,2 °F) but precision is not guaranteed outside of this range.

## TROUBLESHOOTING


If you have problems while using your thermometer, please refer to this guide to help resolve the problem.

- **The screen displays temperature higher than 42,9 °C (109,2°F):** The temperature is in Fahrenheit. Change the measurement to Celsius by pressing °C/°F button.
- **The screen displays temperature lower than 32°C (89,6°F):** o take the surface temperature, press the MODE button and set to the reading called „Body“. If the device is in Surface Temp mode, the temperature displayed is showing the external temperature of your body, rather than the internal.
- **The screen displays the message „HI“:** When using the thermometer, the message „HI“ can appear on the screen. In this case, the temperature is above the measurement range selected, for example above 42,9°C (109,2°F) in Body mode.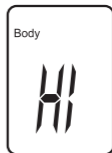

- **The screen displays the message „Lo“:**

When using the thermometer, the message „Lo“ can appear on the screen. In this case, the temperature is above the measurement range selected, for example less than 32 °C (89,6 °F) in Body mode.

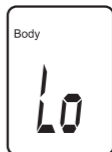This message displays for various reasons:

- push back hair and wipe the make-up from the forehead
- measurement distance is longer than 3-5 cm
- allow a 1 minute interval between two measurements
- avoid any drafts (e.g. from nasal specs, air conditioning...)

**Contents:**

- 1 pc non-contact thermometer Helpmation
- 2 pcs AA, LR6 batteries
- 1 pc instruction manual

This product is in compliance with the essential requirements and other relevant provisions with EU Directives 93/42/EEC.

Do not dispose the product in household waste at the end of its useful life. Please, dispose of this device at your applicable collection point for the recycling of electrical & electronic equipments waste.

Guangzhou Jinxinbao Electronic Co., Ltd.  
Address: No. 38 Huanzhen Xi Road, Dagang Town, Nansha, Guangzhou, China

Wellkang Ltd.  
Address: Suite B, 29 Harley Street London, W1G 9QR, England, United Kingdom

# WARRANTY

Model:

Serial number:

Date of purchase:

Signature and stamp dealer:

The product is guaranteed for 24 months.

The warranty covers defects arising as a result demonstrably defective material, faulty design or bad design.

The warranty does not cover defects caused by transport, normal use of the product (in particular the loss of battery capacity), failure to follow the instructions indicated in the instructions, poor maintenance, mechanical damage, unauthorized intervention in the appliance service. Repairs may only be performed by an authorized service center.

Specifications are based on a condition known at the time of publication. Technical alterations.

BLAKAR trading s.r.o.  
Czech Republic  
[www.helpmation.com](http://www.helpmation.com)